



ASI

The Leader In Advanced Systems Integration
Advanced Dental Systems® & Advanced Endodontic Systems®
www.asimedical.net | (303) 766-3646

Technical Guideline

Installation of Monitor Display Mount

TG-90-1827

Rev A | 04 Aug 2016 | Page 1 of 1

TOOLS REQUIRED

- #2 Phillips head screwdriver

INSTRUCTIONS

1. Place monitor display mount onto carriage.
2. Using a #2 Phillips screwdriver, screw the two flat head screws provided onto the upper carriage. (Fig. 1)



Fig. 1

-
3. Screw the two pan head screws provided onto the front of the carriage.
-