



ASI

The Leader In Advanced Systems Integration
Advanced Dental Systems® & Advanced Endodontic Systems®
www.asimedical.net | (303) 766-3646

Technical Guideline

Installing Handle for Easy Cart Positioning

TG-90-2808

Rev B | 16 Jul 2015 | Page 1 of 1

PARTS INCLUDED

- One (1) handle

TOOLS REQUIRED

- Phillips screwdriver



INSTRUCTIONS

1. Place Handle (90-2808) on Finger Bracket. Attach two (2) 10/32" flat head screws. (Fig. 1)



Fig. 1

Arrow indicates location for the flat head screws.

2. Verify proper and secure installation by moving unit using the Handle.