



The Leader In Advanced Systems Integration
Advanced Dental Systems® & Advanced Endodontic Systems®
www.asimedical.net | (303) 766-3646

Technical Guideline

Accessory Shelf Installation

TG-90-BP07

Rev A | 01/17/17 | Page 1 of 2

PURPOSE

The purpose of this technical guideline is to provide step-by-step instruction for installing an accessory shelf to an ASI delivery system.

PARTS INCLUDED


- Accessory shelf (1)
- Mounting bracket (1)
- End caps (2)
- Compression springs (4)
- Detent Plunger cups (4)
- Flat Head Socket screws (4)
- Phillips Flat Head 6-32 x 1/4" Screws (4)

TOOLS REQUIRED


- Phillips head screwdriver, #2
- 1/8" Allen wrench

INSTRUCTIONS

1. Take one (1) Mounting bracket and insert two (2) Compression springs.

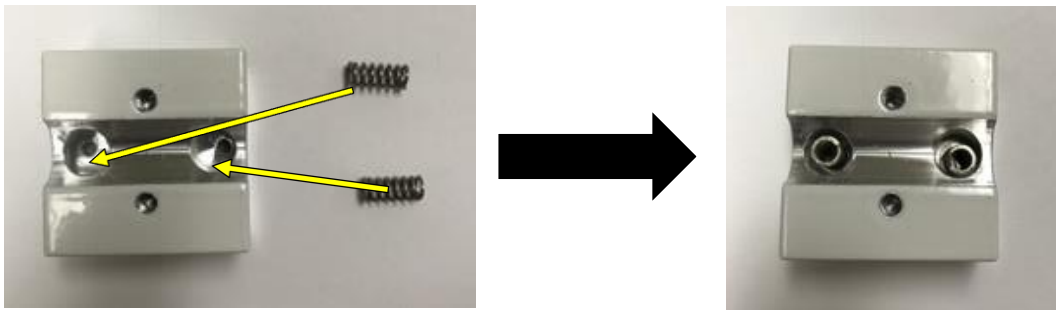

2. Place two (2) Detent Plunger Cups over each spring.

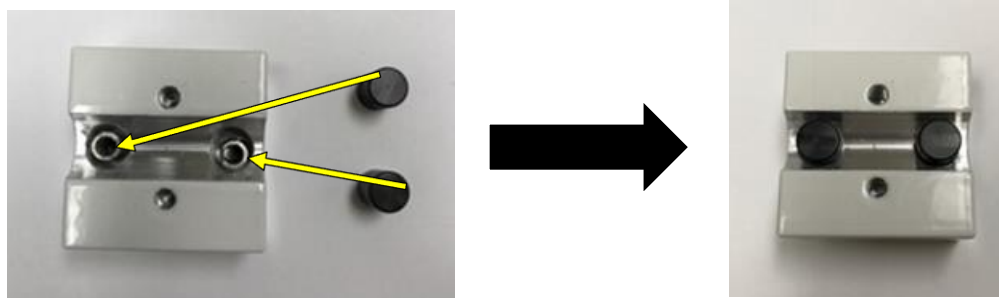



The Leader In Advanced Systems Integration
Advanced Dental Systems® & Advanced Endodontic Systems®
www.asimedical.net | (303) 766-3646

Technical Guideline

Accessory Shelf Installation

TG-90-BP07

Rev A | 01/17/17 | Page 2 of 2


- Using two (2) Flat Head Socket Screws and one (1) End Cap install the Mounting Bracket, from step 2, onto the carts back rail.

NOTE: Do not over tighten the socket head screws, the mounting block should be able to rotate on the rail without excessive force.

Rotate the mounting bracket on the rail so the shelf mounting holes are facing up.



- Repeat Steps 1 thru 3 to add a second Mounting Bracket to the same rail, approximately 7.5" from edge to edge.

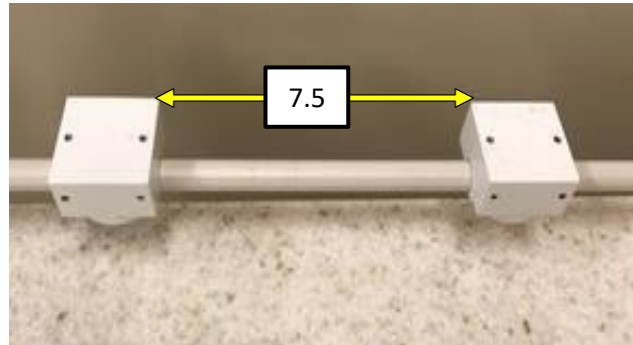

- Using four (4) Phillips Flat Head, 6-32 x 1/4" Screws, install the Accessory Shelf on the two mounting blocks.

