



The Leader In Advanced Systems Integration  
Advanced Dental Systems® & Advanced Endodontic Systems®  
www.asimedical.net | (303) 766-3646

## Technical Guideline

### Battery Replacement for the Coltene Endo Canal Pro

TG-90-BV06

Rev A | 09/14/16 | Page 1 of 2

#### PURPOSE

To guide the end user on how to replace the battery on the Coltene Endo Instrument.



**WARNING!** Only qualified personnel should service or repair this device. This device should only be serviced/ repaired by a qualified service technician who is proficient in the repair of electromechanical dental equipment and who understands the complexities and risks of working within the device and observes proper safety precautions.

#### TOOLS REQUIRED

- 5/32 hex key

#### INSTRUCTIONS

1. Remove socket screw on bottom of the Z bracket. Make sure to hold the Apex locator instrument as the screw you removing is what hold it. (Fig.1)



Fig. 1



The Leader In Advanced Systems Integration  
Advanced Dental Systems® & Advanced Endodontic Systems®  
www.asimedical.net | (303) 766-3646

## Technical Guideline

### Battery Replacement for the Coltene Endo Canal Pro

TG-90-BV06

Rev A | 09/14/16 | Page 2 of 2

2. Take black plastic space and set a side. (Fig. 2)



Fig. 2

3. Unscrew the plastic cover plate from Apex locator to access the battery. (Fig. 3)

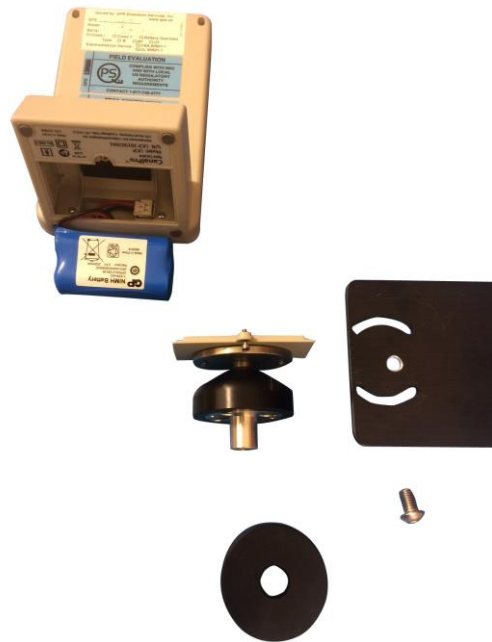


Fig. 3

4. Reverse the steps to reassemble.