



# ASI

The Leader In Advanced Systems Integration  
Advanced Dental Systems® & Advanced Endodontic Systems®  
www.asimedical.net | (303) 766-3646

## Technical Guideline

### Replacing Luer Lock Ring Finger Snap Adapter

TG-95-0318

Rev B | 28 Oct 2015 | Page 1 of 1

#### PARTS INCLUDED

- Ring adapters, Luer lock (5)

#### TOOLS REQUIRED

- Small flat head screwdriver

#### INSTRUCTIONS

##### Attaching Optional Luer Lock Adapter

Slide locking adapter (threaded end up) onto the of the handpiece shaft. There is a ridge at the base of the shaft that the ring needs to slide over. The ring will snap into place when the ring is pushed down enough. (Fig. 1)



Fig. 1

##### Replacing Luer Lock Adapter

**NOTE:** Replace the Luer lock adapter if needed.

To remove and replace the Luer ring adapter, use a small flat head screwdriver at the base of the shaft and twist it to dislodge the adapter from the ridge on the slip Luer shaft. (Fig. 2)

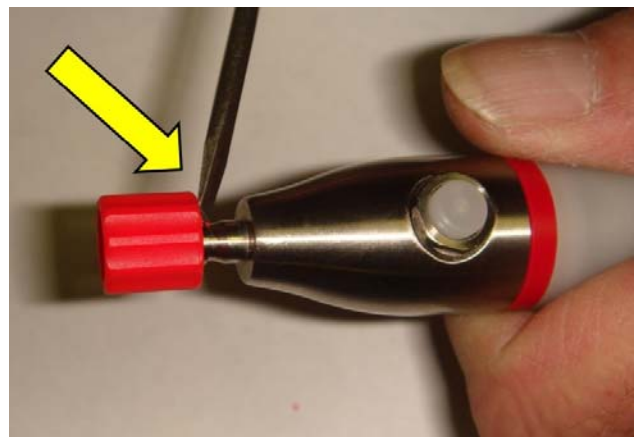


Fig. 2