PREVENTATIVE MAINTENANCE

ANNUAL & 5-YEAR EVALUATION CHECKLIST

65-0244 | Rev C | 10/13/22 | Page 1 of 5

DELIVERY SYSTEMS

- ___ Verify handpiece pressure gauge is operational. Check for crack or air leaks. Replace, if needed (PN 95-0136).
- ___ Run a handpiece and verify operation of water relay valve for opening and shutting off. Recommended replacement every 5 years (PN 95-0135).
- ___ Test all handpieces for proper air pressure, water function. Using the stems on the control block, make adjustments. Replace control block diaphragms every 5 years (PN 95-0215).
- ___ Check all gaskets in the handpiece tubing bulkheads. Replace every 5 years (PN 95-0206).
- ___ Inspect and replace handpiece tubing if worn, brittle, or hardened.
- ___ Verify operation of auto holders that actuator ball freely moves in and out. Clean with hot water and paper towel if necessary to remove any debris.

FOOT PEDAL

- ___ Lubricate O-rings and poppet valve using silicone grease (PN 95-0085).
- ___ Install repair kit every 5 years. Serial Number specific:
 - PN 95-0094 [SN 595368 and earlier]
 - PN 95-0240 [SN 595369 and later]
- ___ Check water toggle for proper function. Replace, if needed (PN 95-0242).

CLOSED WATER SYSTEM

- ___ Turn air pressure toggle to ON. Verify closed water system gauge is operational. Check for cracks or air leaks. Replace gauge, if needed (PN 95-0266).
- ___ Verify air pressure is 25-35 psi. If high or low, inspect inline regulator by pressing in on both ends. If pressure does not adjust within pressure setting, replace regulator (PN 95-0216).
- ___ Replace water bottles annually
 - PN 95-0347 for 38mm neck size
 - PN 95-0140 for 28mm neck size
- ___ Replace DentaPure disinfection cartridges annually (PN 95-0281).

AIR/WATER SYRINGE

- ___ Inspect tubing for wear, kinks and damage. Replace if needed every 5 years.
 - PN 95 0213 [60" Length]
 - PN 95-0214 [50" Length]
- ___ Verify operation of air and water buttons and absence of leaks. Install syringe repair kit (PN 95-0089), if necessary. Recommended replacement every 1-3 years.

PREVENTATIVE MAINTENANCE

ASI Technical Guideline for Annual & 5-year Evaluation Checklist

65-0244 | Rev C | 10/13/22 | Page 2 of 5

FIBER OPTICS AND OTHER ELECTRIC INSTRUMENTS

- ___ Check integrity of fiber optic tubing if worn, brittle or hardened. Inspect wires for break and proper connection to power pack. Replace tubing, fiber optic bulb, power pack, if needed.
 - PN 95-0114W [60" Length]
 - PN 95-0265W [50" Length]
- ___ Check integrity of other electric instrument cords and repair or replace if jacket is worn or torn.
- ___ Replace air electric switches, if included, every 5 years:
 - PN 95-0137 [Normally Open]
 - PN 95-0138 [Normally Closed]

OPTIONAL SUCTION INSTRUMENTS AND ACCESSORIES

- ___ Inspect solids trap lid O-ring for wear. Lubricate using silicone O-ring grease (95-0085). Replace if needed (PN 95-0091L).
- ___ Inspect solids trap housing for cracks. Replace, if needed.
 - PN 95-0247 [Side Port]
- ___ Inspect all suction tubing for leaks or damage. Replace, if needed.
 - PN 95-0287 [HVE]
 - PN 95-0288 [Saliva Ejector]
- ___ Replace suction lever O-rings and connector barb O-rings (PN 95-4040 and 95-4034).

FOR UMBILICAL UNITS ONLY

- ___ Verify air pressure from main air connection is 75-80 psi; adjust if needed. ASI units risk damage if air pressure is set too high and will void the warranty.

FOR SELF-CONTAINED UNITS ONLY - VACUUM PUMP SYSTEM

- ___ Empty sediment bowl per suction canister maintenance.
- ___ Replace hose assembly/waste purge, peristaltic every 5 years:
 - PN 95-0205 [Side Drain]
 - PN 95-0334 [Bottom Drain]
- ___ Inspect all vacuum tubing and fittings for leaks or damage. Replace, if needed.
- ___ Replace felt filter muffler (PN 95-0093).
- ___ Install rebuild kit for vacuum pump every 5-10 years, if needed. Serial Number specific:
 - PN 95-0068 [SN 180319 and earlier]
 - PN 95-0378 [SN 180320 and later]
- ___ Test for proper function of purge pump and motor. Replace, if necessary, every 5 years (PN 95-0293).

- ___ Inspect exterior waste purge hose assembly. If worn, brittle, or hardened, replace every 5 years. Serial Number specific:
 - PN 95-0306 [SN 200101 and earlier]
 - PN 95-0401 [SN 200201 and later]
- ___ Clean and/or replace inline purge filter (PN 95-0423) [SN 200934 and later]

FOR SELF-CONTAINED UNITS ONLY - INTERNAL AIR COMPRESSOR SYSTEM

- ___ Inspect master air regulator for air leaks, proper function. Replace every 10 years, if needed. Serial Number specific:
 - Upgrade to PN 95-0345 [SN 170100 and earlier]
 - PN 95-0178A [SN 170101 and later]
 - PN 95-0178C [SN 190819 and later]
- ___ Inspect filter regulator setting and make sure it reads 75-80 psi, adjust, if needed.
- ___ Inspect tubing and replace, if needed.
- ___ Replace felt filter muffler (PN 95-0093).
- ___ Install rebuild kit for compressor every 5-10 years, if needed. Serial Number specific:
 - PN 95-0069 [SN 180319 and earlier]
 - PN 95-0378 [SN 180320 and later]
- ___ Inspect and clean check valve. Replace, if needed (PN 95-0108).
- ___ Test Compressor pressure switch for proper function. Replace, if needed (PN 95-0264).

WARNINGS/CAUTIONS

In addition to observing the normal precautions associated with standard dental practices and procedures, the following additional precautions should be strictly noted and observed during the set-up, operation, and maintenance of this system.

WARNING

QUALIFIED PERSONNEL ONLY

The product should only be operated by qualified personnel only. The operator bears responsibility for the correct settings and proper use of the system. ASI Dental (ASI) cannot be held liable for any malfunction of this product, or performance failure and/or its designed or desired utility, nor can ASI be held liable for injuries to persons or animals, in any case when the device is misused or not operated, applied or maintained in strict accordance with user/owner instructions set out in the operation manual. In the event of any doubt or question, the user is to contact ASI for clarification or assistance.

Improperly maintained or operated systems or instruments may void the associated warranties.

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PREVENTATIVE MAINTENANCE

ASI Technical Guideline for Annual & 5-year Evaluation Checklist

65-0244 | Rev C | 10/13/22 | Page 4 of 5

WARNING

COMPRESSED AIR

The compressed air system that operates this unit is under pressure. Compressed air can propel dust or loose particles and can cause bodily injury or damage. Always turn the system off and bleed off air pressure before attaching or removing air lines or accessories or servicing this unit. All air lines should be periodically inspected and replaced if worn or damaged.

If an outside compressed air supply is used to power this unit, the air supply must be regulated to 80 psi or below. Excessive air pressure could cause certain components to rupture.

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WARNING

ELECTRICAL VOLTAGE

This system is powered by high voltage electricity. Like any other electrically powered device, if it is not used properly, it can cause electrical shock. Always plug the power cord into an electrical outlet with adequate fuse protection and proper grounding. In the event of a short circuit, grounding reduces the risk of shock by providing an escape wire for the electric current. Improper grounding of the unit can result in a risk of electric shock. Always unplug the unit before doing any service or repair to the unit.

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WARNING

PRESENCE OF HEAVY METALS/AMALGAM

This system may be equipped with optional suction instruments. As part of dental procedures, particles of amalgam may be suctioned into the dental suction handpieces and collected within the system and trap filter.

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WARNING

INFECTIOUS MATERIALS

Infectious disease workplace safety protocols to safeguard against cross contamination of infectious disease should always be observed. When maintaining the suction system or emptying the contents of the suction waste container, safe precautions and practices including the wearing of face mask, eye protection and gloves are to be followed.

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CAUTION

CAUSTIC OR CORROSIVE CHEMICALS

The irrigation system may contain chemicals that are caustic or corrosive. Always verify with the operator of the system to determine which chemicals they are using within the system. Always follow workplace safety protocols when doing maintenance or service of the system including the wearing of face mask, eye protection and gloves for proper protection of exposed skin, breathing and eyes.

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CAUTION

DAMAGE FROM UNSUITABLE ACCESSORIES

Responsibility for the use of accessories, parts or assemblies from other manufacturers rest solely with the user.

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