# PERISTALTIC PURGE PUMP

## ASI TECHNICAL GUIDELINE FOR 95-0419

65-0143 | Rev A1 | 05/03/23 | Page 1 of 5

### PURPOSE

This technical guideline provides instructions for installing a peristaltic purge pump in ASI delivery systems manufactured after September 2019.

- 95-0419, Peristaltic Purge Pump

### PARTS INCLUDED

- Purge pump

### TOOLS REQUIRED

- Flat head screwdriver
- #2 Phillips head screwdriver
- Wire crimpers
- 5/16" wrench

#### IMPORTANT!

**Before commencing on the following instructions, please read and follow all applicable warnings/cautions listed at the end of this technical guideline.**

### INSTRUCTIONS

- 1 Turn off power to the unit.
- 2 Open back panel of unit using a flat head screwdriver. There are two quick-turn screws on the back of unit. Move suction tube and electrical cord out of the way of back door. (Fig. 1)

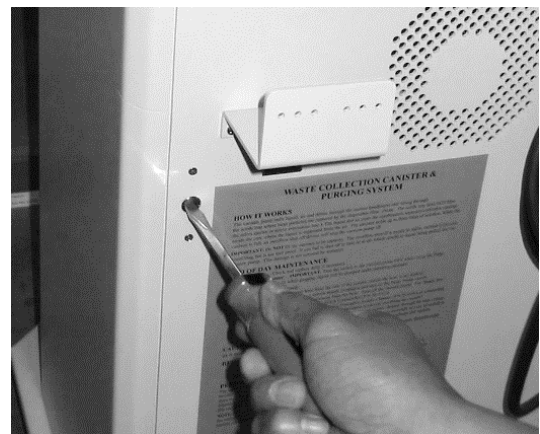
Fig. 1

# PERISTALTIC PURGE PUMP

ASI Technical Guideline for 95-0419

65-0143 | Rev A1 | 05/03/23 | Page 2 of 5

- 3 Locate peristaltic purge pump housing and open to access the tubing. (Fig. 2)

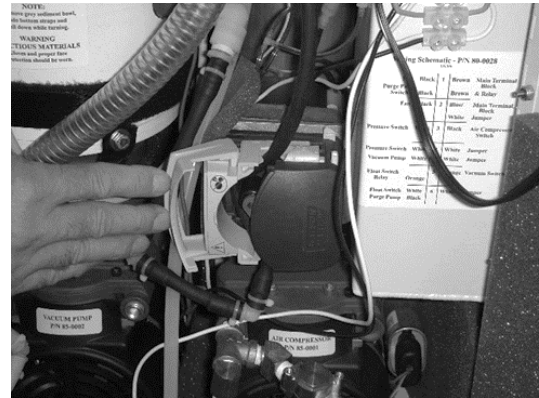
Fig. 2

- 4 Remove the existing black pump tubing from the housing. (Fig. 3)

Fig. 3

- 5 Insert the head of a flat head screwdriver between the white plastic adapter and the white plastic connector on the tubing. Depress white ring while pulling down on the tubing. (Fig. 4)

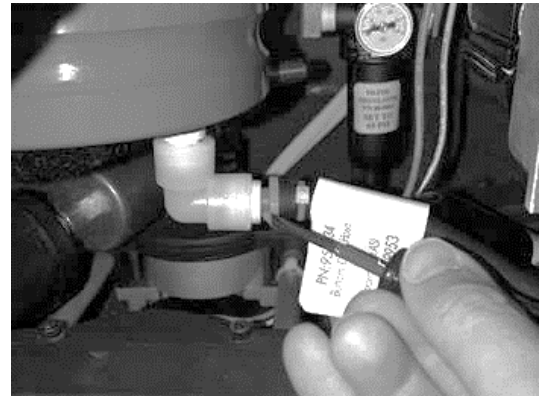
Fig. 4

- 6 Cut the zip tie and remove tubing from male end of purge connection. (Fig. 5)

Fig. 5

- 7 Replace with new tubing assembly and follow the instructions in reverse to complete the install.
- 8 Turn system on and suction up a small amount of clean water. Then, turn on purge pump switch to make sure it is purging properly and there are no leaks in new tubing assembly.

# PERISTALTIC PURGE PUMP

ASI Technical Guideline for 95-0419

65-0143 | Rev A1 | 05/03/23 | Page 4 of 5

## Purge Pump Removal/Replacement

- 9 Disconnect the bullet connectors from the purge pump wires. (Fig. 6)

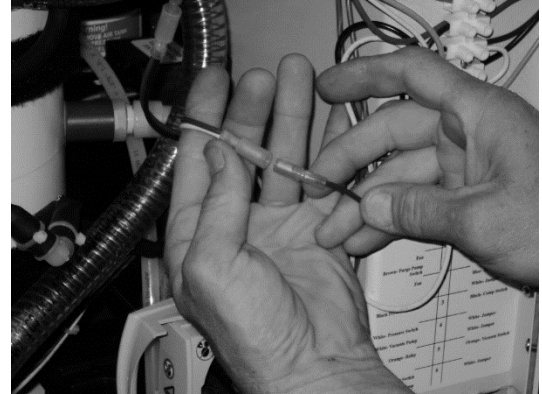
Fig. 6

- 10 Unscrew the purge pump from the water housing. The four screws that hold the pump are accessible from the left door of the cart. (Fig. 7)

Fig. 7

- 11 Crimp the bullet connectors the exact same way that are on the wires of the old purge pump.
- 12 Install the new purge pump by reversing the above steps. Install new purge tubing; steps 1 through 8 reversed.
- 13 Turn system on and suction up a small amount of clean water. Then, turn on purge pump switch to make sure it is purging properly and that there are no leaks.

## WARNINGS/CAUTIONS

In addition to observing the normal precautions associated with standard dental practices and procedures, the following additional precautions should be strictly noted and observed during the set-up, operation, and maintenance of this system.

### WARNING

#### QUALIFIED PERSONNEL ONLY

The product should only be operated by qualified personnel only. The operator bears responsibility for the correct settings and proper use of the system. ASI Dental (ASI) cannot be held liable for any malfunction of this product, or performance failure and/or its designed or desired utility, nor can ASI be held liable for injuries to persons or animals, in any case when the device is misused or not operated, applied or maintained in strict accordance with user/owner instructions set out in the operation manual. In the event of any doubt or question, the user is to contact ASI for clarification or assistance.

Improperly maintained or operated systems or instruments may void the associated warranties.

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### WARNING

#### ELECTRICAL VOLTAGE

This system is powered by high voltage electricity. Like any other electrically powered device, if it is not used properly, it can cause electrical shock. Always plug the power cord into an electrical outlet with adequate fuse protection and proper grounding. In the event of a short circuit, grounding reduces the risk of shock by providing an escape wire for the electric current. Improper grounding of the unit can result in a risk of electric shock. Always unplug the unit before doing any service or repair to the unit.

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### WARNING

#### INFECTIOUS MATERIALS

Infectious disease workplace safety protocols to safeguard against cross contamination of infectious disease should always be observed. When maintaining the suction system or emptying the contents of the suction waste container, safe precautions and practices including the wearing of face mask, eye protection and gloves are to be followed.

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