WATER BOTTLE GASKETS – 28mm & 38mm

ASI TECHNICAL GUIDELINE FOR 95-0146 & 95-0156

65-0181 | Rev B1 | 05/12/23 | Page 1 of 4

PURPOSE

This technical guideline provides instructions to replace gaskets in 28mm and 38mm water bottles, used in ASI delivery systems.

- 95-0146, Gasket 28MM-Pressure Head .75"ID x 1.125"OD x .062"
- 95-0156, Gasket 38MM, Pressure Head 1" ID x 1.5" OD x .062"

PARTS INCLUDED

- Gaskets (10)

TOOLS REQUIRED

- Flat head screwdriver

IMPORTANT!

Before commencing on the following instructions, please read and follow all applicable warnings/cautions listed at the end of this technical guideline.

INSTRUCTIONS

- 1 Turn off air supply to bottle housings.
- 2 Slightly unscrew a bottle to relieve compressed air in system.
- 3 Remove existing bottles from bracket. (Fig. 1)

Fig. 1

WATER BOTTLE GASKETS – 28mm & 38mm

ASI Technical Guideline for 95-0146 & 95-0156

65-0181 | Rev B1 | 05/12/23 | Page 2 of 4

- 4 Look up inside of pressure head and locate the gaskets. (Fig. 2)

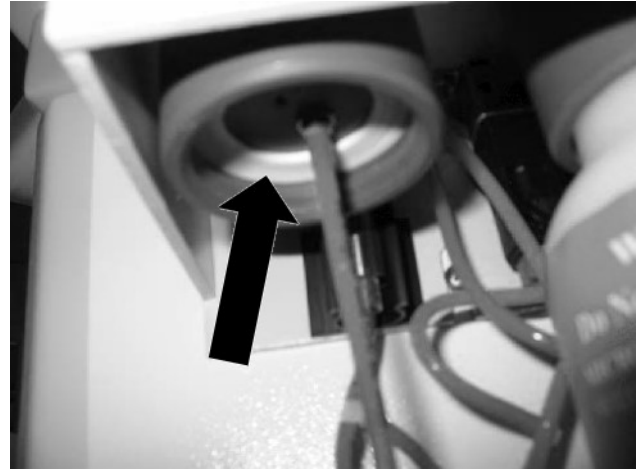
Fig. 2

- 5 Using a flathead screwdriver, remove the gasket from the pressure head. (Fig. 3)

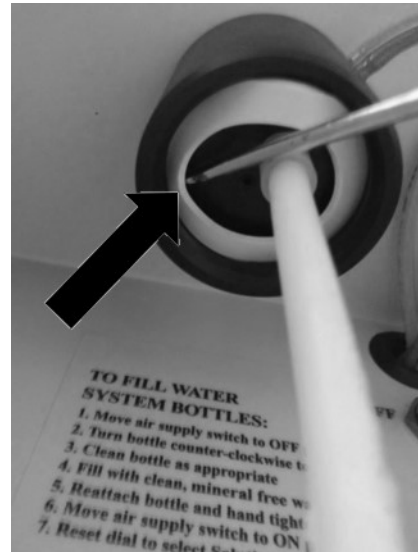
Fig. 3

- 6 Insert new gasket in the same place.
 - i **Make sure the gasket is seated properly with no tears or wrinkles. You may need to walk the gasket around with your finger to seat into the groove. (Fig. 4)**

Fig. 4

- 7 Fill bottles with water and rethread to pressure heads.
- 8 Restore power to the unit and verify proper installation. For water systems, test water flow from bottle #1 and bottle #2 by flipping the valve back and forth while running the water button on the air/water syringe. For Irrigation Systems, test solution flow by running irrigation instrument(s).

WARNINGS/CAUTIONS

In addition to observing the normal precautions associated with standard dental practices and procedures, the following additional precautions should be strictly noted and observed during the set-up, operation, and maintenance of this system.

WARNING

QUALIFIED PERSONNEL ONLY

The product should only be operated by qualified personnel only. The operator bears responsibility for the correct settings and proper use of the system. ASI Dental (ASI) cannot be held liable for any malfunction of this product, or performance failure and/or its designed or desired utility, nor can ASI be held liable for injuries to persons or animals, in any case when the device is misused or not operated, applied or maintained in strict accordance with user/owner instructions set out in the operation manual. In the event of any doubt or question, the user is to contact ASI for clarification or assistance.

Improperly maintained or operated systems or instruments may void the associated warranties.

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WARNING

COMPRESSED AIR

The compressed air system that operates this unit is under pressure. Compressed air can propel dust or loose particles and can cause bodily injury or damage. Always turn the system off and bleed off air pressure before attaching or removing air lines or accessories or servicing this unit. All air lines should be periodically inspected and replaced if worn or damaged.

If an outside compressed air supply is used to power this unit, the air supply must be regulated to 80 psi or below. Excessive air pressure could cause certain components to rupture.

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WARNING

ELECTRICAL VOLTAGE

This system is powered by high voltage electricity. Like any other electrically powered device, if it is not used properly, it can cause electrical shock. Always plug the power cord into an electrical outlet with adequate fuse protection and proper grounding. In the event of a short circuit, grounding reduces the risk of shock by providing an escape wire for the electric current. Improper grounding of the unit can result in a risk of electric shock. Always unplug the unit before doing any service or repair to the unit.

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CAUTION

CAUSTIC OR CORROSIVE CHEMICALS

The irrigation system may contain chemicals that are caustic or corrosive. Always verify with the operator of the system to determine which chemicals they are using within the system. Always follow workplace safety protocols when doing maintenance or service of the system including the wearing of face mask, eye protection and gloves for proper protection of exposed skin, breathing and eyes.

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