FINGER MOUNT HOLDER BAR BRACKET

ASI TECHNICAL GUIDELINE FOR 95-0319

65-0205 | Rev B1 | 05/19/23 | Page 1 of 3

PURPOSE

This technical guideline provides instructions to install a finger mount holder bar bracket on ASI delivery units manufactured prior to July 2011. Finger mount accessory brackets were not used prior to this date.

PARTS INCLUDED

- 95-0319, Finger mount holder bar bracket (shown)

TOOLS REQUIRED

- Phillips screwdriver
- 1/4" nut driver or short 1/4" crescent wrench
- 10/32" drill bit
- Power drill
- Felt-tip marker or pencil

IMPORTANT!

Before commencing on the following instructions, please read and follow all applicable warnings/cautions listed at the end of this technical guideline.

INSTRUCTIONS

- 1 Turn off main power and air supply to unit.
- 2 Open top of unit cover; refer to the operation instructions manual for your model.
- 3 Remove handpiece pressure gauge for drill access. Remove two nuts holding gauge to unit using 1/4" nut driver or small 1/4" crescent wrench. (Fig. 1)

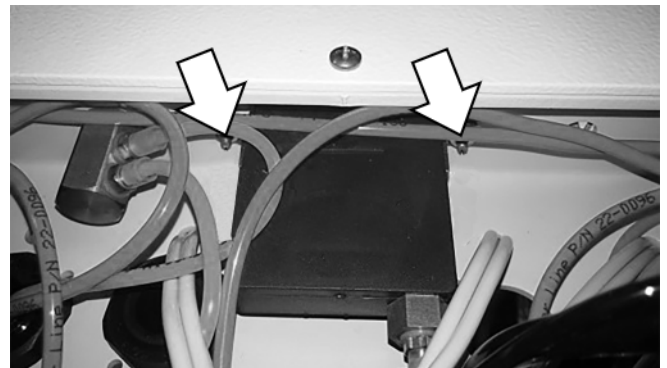
Fig. 1

FINGER MOUNT HOLDER BAR BRACKET

ASI Technical Guideline for 95-0319

65-0205 | Rev B1 | 05/19/23 | Page 2 of 3

- 4 Place finger bracket on holder bar and align with underside of baseplate. Mark positions of screw holes using felt-tip pen or pencil. Using 10/32" bit, drill holes at marked locations. (Fig. 2)

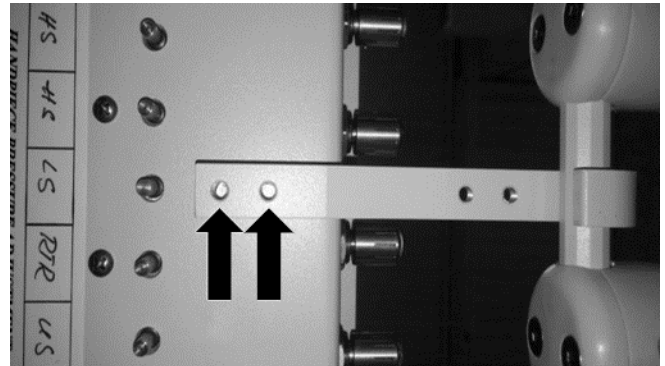
Fig. 2

- 5 Attach bar using two (2) 10x32" round head screws through drilled holes. (Fig. 3)

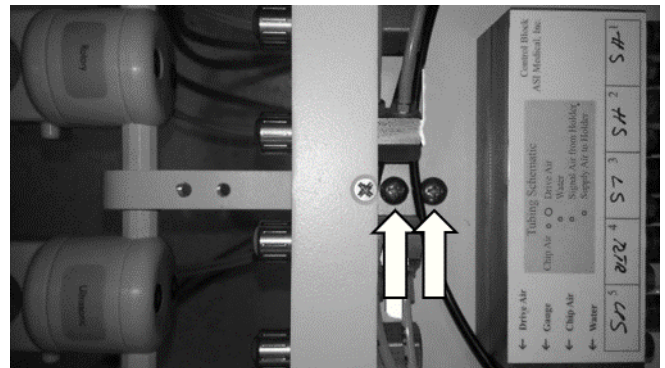
Fig. 3

- 6 Reinstall pressure gauge.
- 7 Verify proper and secure installation by moving unit using the handle.

WARNINGS/CAUTIONS

In addition to observing the normal precautions associated with standard dental practices and procedures, the following additional precautions should be strictly noted and observed during the set-up, operation, and maintenance of this system.

WARNING

QUALIFIED PERSONNEL ONLY

The product should only be operated by qualified personnel only. The operator bears responsibility for the correct settings and proper use of the system. ASI Dental (ASI) cannot be held liable for any malfunction of this product, or performance failure and/or its designed or desired utility, nor can ASI be held liable for injuries to persons or animals, in any case when the device is misused or not operated, applied or maintained in strict accordance with user/owner instructions set out in the operation manual. In the event of any doubt or question, the user is to contact ASI for clarification or assistance.

Improperly maintained or operated systems or instruments may void the associated warranties.

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WARNING

COMPRESSED AIR

The compressed air system that operates this unit is under pressure. Compressed air can propel dust or loose particles and can cause bodily injury or damage. Always turn the system off and bleed off air pressure before attaching or removing air lines or accessories or servicing this unit. All air lines should be periodically inspected and replaced if worn or damaged.

If an outside compressed air supply is used to power this unit, the air supply must be regulated to 80 psi or below. Excessive air pressure could cause certain components to rupture.

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WARNING

ELECTRICAL VOLTAGE

This system is powered by high voltage electricity. Like any other electrically powered device, if it is not used properly, it can cause electrical shock. Always plug the power cord into an electrical outlet with adequate fuse protection and proper grounding. In the event of a short circuit, grounding reduces the risk of shock by providing an escape wire for the electric current. Improper grounding of the unit can result in a risk of electric shock. Always unplug the unit before doing any service or repair to the unit.

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