# WATER BOTTLES, 38mm

## ASI TECHNICAL GUIDELINE FOR 95-0347

65-0210 | Rev D1 | 05/19/23 | Page 1 of 3

### PURPOSE

This technical guideline shows the user how to replace the water bottle (38mm diameter mouth), for delivery units that were manufactured after 06/30/2011 (or after serial number 594559).

### PARTS INCLUDED

- Water bottle(s)

### TOOLS REQUIRED

- Flat head screwdriver or hemostat


#### IMPORTANT!

**Before commencing on the following instructions, please read and follow all applicable warnings/cautions listed at the end of this technical guideline.**

### INSTRUCTIONS

- 1 Turn off main power and air supply to water bottle system.
- 2 Depressurize bottles and unthread from pressure head.
- 3 Remove gasket from pressure head using a flat head screwdriver or hemostats. (Fig. 1)

**i** *Gaskets are no longer needed for the closed water bottle system with quick-turn adapter bottles.*

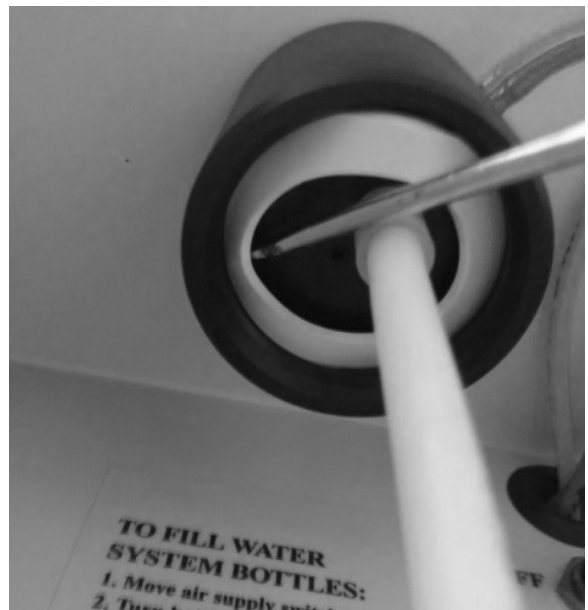# WATER BOTTLES, 38mm

ASI Technical Guideline for 95-0347

65-0210 | Rev D1 | 05/19/23 | Page 2 of 3

- 4 Fill bottle with water and rethread onto pressure head. (Fig. 2, Fig. 3)

Fig. 2

Fig. 3

- 5 Restore power to unit, re-pressurize the water bottle system, and verify correct installation by testing handpieces for proper water function.

## WARNINGS/CAUTIONS

In addition to observing the normal precautions associated with standard dental practices and procedures, the following additional precautions should be strictly noted and observed during the set-up, operation, and maintenance of this system.

### ⚠ WARNING

#### QUALIFIED PERSONNEL ONLY

The product should only be operated by qualified personnel only. The operator bears responsibility for the correct settings and proper use of the system. ASI Dental (ASI) cannot be held liable for any malfunction of this product, or performance failure and/or its designed or desired utility, nor can ASI be held liable for injuries to persons or animals, in any case when the device is misused or not operated, applied or maintained in strict accordance with user/owner instructions set out in the operation manual. In the event of any doubt or question, the user is to contact ASI for clarification or assistance.

Improperly maintained or operated systems or instruments may void the associated warranties.

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## WARNING

### COMPRESSED AIR

The compressed air system that operates this unit is under pressure. Compressed air can propel dust or loose particles and can cause bodily injury or damage. Always turn the system off and bleed off air pressure before attaching or removing air lines or accessories or servicing this unit. All air lines should be periodically inspected and replaced if worn or damaged.

If an outside compressed air supply is used to power this unit, the air supply must be regulated to 80 psi or below. Excessive air pressure could cause certain components to rupture.

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