TANDEM DUAL CLOSED WATER SYSTEM UPGRADE

ASI TECHNICAL GUIDELINE FOR 95-0382 & 95-0389

65-0214 | Rev C1 | 05/19/23 | Page 1 of 3

PURPOSE

This technical guideline provides instructions for how to replace a bottle selector style dual closed water system (DCWS) and installing the tandem DCWS.

PARTS INCLUDED

- Tandem DCWS bracket (95-0382)
- Tube coupler (x2)
- Tube sleeve (x4)
- **i** *95-0389 includes a DentaPure disinfection cartridge and two new bottles.*

TOOLS REQUIRED

- Phillips screwdriver #2
- 3/8" wrench (socket)
- Tube cutter
- Sleeve tool

IMPORTANT!

Before commencing on the following instructions, please read and follow all applicable warnings/cautions listed at the end of this technical guideline.

INSTRUCTIONS

- 1 Turn off main power. Unplug unit from outlet and air supply to unit.
- 2 Remove the two water bottles from the DCWS.

Fig. 1

TANDEM DUAL CLOSED WATER SYSTEM UPGRADE

ASI Technical Guideline for 95-0382 & 95-0389

65-0214 | Rev C1 | 05/19/23 | Page 2 of 3

- 3 Depending on the system you have, remove the two Phillips head screws or the two locknuts attaching the bottle bracket.

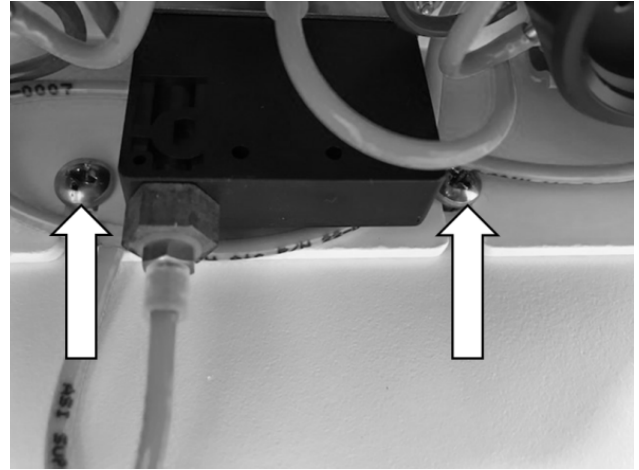
Fig. 2

- 4 Cut the yellow (air) and blue (water) tubing connecting the DCWS to the cart.

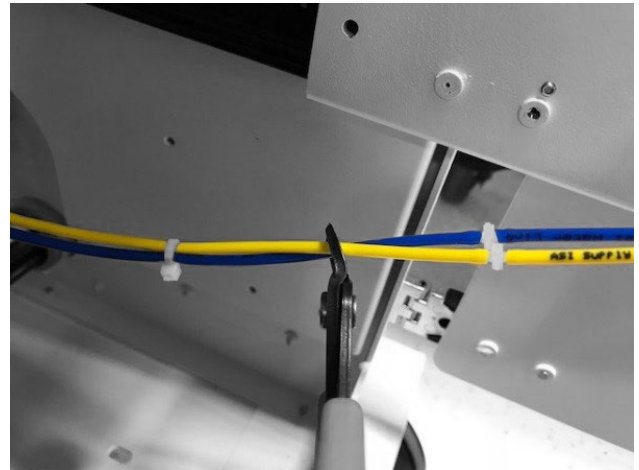
Fig. 3

- 5 Install the tandem DCWS, reinstalling the two screws or two locknuts.
- 6 Then connect the tubing from the tandem DCWS to the cart tubing using the two tube couplers and sleeves provided.

WARNINGS/CAUTIONS

In addition to observing the normal precautions associated with standard dental practices and procedures, the following additional precautions should be strictly noted and observed during the set-up, operation, and maintenance of this system.

WARNING

QUALIFIED PERSONNEL ONLY

The product should only be operated by qualified personnel only. The operator bears responsibility for the correct settings and proper use of the system. ASI Dental (ASI) cannot be held liable for any malfunction of this product, or performance failure and/or its designed or desired utility, nor can ASI be held liable for injuries to persons or animals, in any case when the device is misused or not operated, applied or maintained in strict accordance with user/owner instructions set out in the operation manual. In the event of any doubt or question, the user is to contact ASI for clarification or assistance.

Improperly maintained or operated systems or instruments may void the associated warranties.

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WARNING

COMPRESSED AIR

The compressed air system that operates this unit is under pressure. Compressed air can propel dust or loose particles and can cause bodily injury or damage. Always turn the system off and bleed off air pressure before attaching or removing air lines or accessories or servicing this unit. All air lines should be periodically inspected and replaced if worn or damaged.

If an outside compressed air supply is used to power this unit, the air supply must be regulated to 80 psi or below. Excessive air pressure could cause certain components to rupture.

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