# MICRO EVACUATOR RETROFIT KIT

## ASI TECHNICAL GUIDELINE FOR 95-0411

65-0220 | Rev A1 | 06/30/23 | Page 1 of 3

### PURPOSE

This technical guideline provides instructions to install a micro evacuator retrofit kit (1/4" to 3/8") in ASI dental systems.

### PARTS INCLUDED

- Micro evacuator refit kit (PN 95-0411)

### TOOLS REQUIRED

- 6" crescent wrench

#### **IMPORTANT!**

**Before commencing on the following instructions, please read and follow all applicable warnings/cautions listed at the end of this technical guideline.**

### INSTRUCTIONS

- 1 Turn suction off to the cart.
- 2 Open the cover plate and disconnect 1/4" suction tubing from micro evacuator handpiece tubing.
- 3 From the end of umbilical pull 1/4" suction tubing out and replace with 3/8" tubing provided.
- 4 Replace micro evacuator handle and tubing with new handle and 3/8" provided.
- 5 Couple handle tubing and suction tubing from umbilical inside the cart, with coupler provided.
- 6 Replace suction adapter from junction box with 3/8" adapter provided.
- 7 Connection suction tubing to adapter.
- 8 Close cover and turn suction on.

# MICRO EVACUATOR RETROFIT KIT

ASI Technical Guideline for 95-0411

65-0220 | Rev A1 | 06/30/23 | Page 2 of 3

## WARNINGS/CAUTIONS

In addition to observing the normal precautions associated with standard dental practices and procedures, the following additional precautions should be strictly noted and observed during the set-up, operation, and maintenance of this system.

### WARNING

#### QUALIFIED PERSONNEL ONLY

The product should only be operated by qualified personnel only. The operator bears responsibility for the correct settings and proper use of the system. ASI Dental (ASI) cannot be held liable for any malfunction of this product, or performance failure and/or its designed or desired utility, nor can ASI be held liable for injuries to persons or animals, in any case when the device is misused or not operated, applied or maintained in strict accordance with user/owner instructions set out in the operation manual. In the event of any doubt or question, the user is to contact ASI for clarification or assistance.

Improperly maintained or operated systems or instruments may void the associated warranties.

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### WARNING

#### COMPRESSED AIR

The compressed air system that operates this unit is under pressure. Compressed air can propel dust or loose particles and can cause bodily injury or damage. Always turn the system off and bleed off air pressure before attaching or removing air lines or accessories or servicing this unit. All air lines should be periodically inspected and replaced if worn or damaged.

If an outside compressed air supply is used to power this unit, the air supply must be regulated to 80 psi or below. Excessive air pressure could cause certain components to rupture.

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### WARNING

#### ELECTRICAL VOLTAGE

This system is powered by high voltage electricity. Like any other electrically powered device, if it is not used properly, it can cause electrical shock. Always plug the power cord into an electrical outlet with adequate fuse protection and proper grounding. In the event of a short circuit, grounding reduces the risk of shock by providing an escape wire for the electric current. Improper grounding of the unit can result in a risk of electric shock. Always unplug the unit before doing any service or repair to the unit.

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## WARNING

### **PRESENCE OF HEAVY METALS/AMALGAM**

This system may be equipped with optional suction instruments. As part of dental procedures, particles of amalgam may be suctioned into the dental suction handpieces and collected within the system and trap filter.

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## WARNING

### **INFECTIOUS MATERIALS**

Infectious disease workplace safety protocols to safeguard against cross contamination of infectious disease should always be observed. When maintaining the suction system or emptying the contents of the suction waste container, safe precautions and practices including the wearing of face mask, eye protection and gloves are to be followed.

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## CAUTION

### **CAUSTIC OR CORROSIVE CHEMICALS**

The irrigation system may contain chemicals that are caustic or corrosive. Always verify with the operator of the system to determine which chemicals they are using within the system. Always follow workplace safety protocols when doing maintenance or service of the system including the wearing of face mask, eye protection and gloves for proper protection of exposed skin, breathing and eyes.

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