MASTER AIR REGULATOR WITH PURGE & MANIFOLD

ASI TECHNICAL GUIDELINE FOR 95-0178C

65-0189 | Rev A1 | 05/19/23 | Page 1 of 4

PURPOSE

This technical guide shows how to replace the master air regulator with manifold on a self-contained cart.

PARTS INCLUDED

- 95-0178C, Master Air Regulator with Purge & Manifold

TOOLS REQUIRED

- Flat head screwdriver

IMPORTANT!

Before commencing on the following instructions, please read and follow all applicable warnings/cautions listed at the end of this technical guideline.

INSTRUCTIONS

- 1 Depressurize air tanks and then turn off the system and unplug from the power outlet.
- 2 Open the back panel using a flat head screwdriver to release the two cam latches, by turning counter-clockwise.
- 3 Locate the pressure regulator near the top right of the inner compartment.

Then loosen and remove the large plastic nut that is holding the regulator in the bracket; see Fig. 1.

Fig. 1 – Master Air Regulator

MASTER AIR REGULATOR WITH PURGE & MANIFOLD

ASI Technical Guideline for 95-0178C

65-0189 | Rev A1 | 05/19/23 | Page 2 of 4

- 4 Release the tubing connected to the push-to-connect fittings on the regulator.

i *Two are located on the bottom of the regulator filter bowl and one is located on the back of the regulator (towards the front of the cart); see Fig. 2.*

Fig. 2 – Push-to-Connect Fittings

MASTER AIR REGULATOR WITH PURGE & MANIFOLD

ASLT-AIR-REGUL-05-01706

- 5 Release the manifold from the quick-disconnect fitting, by pressing on the tab on the quick-disconnect socket and pulling out the manifold.

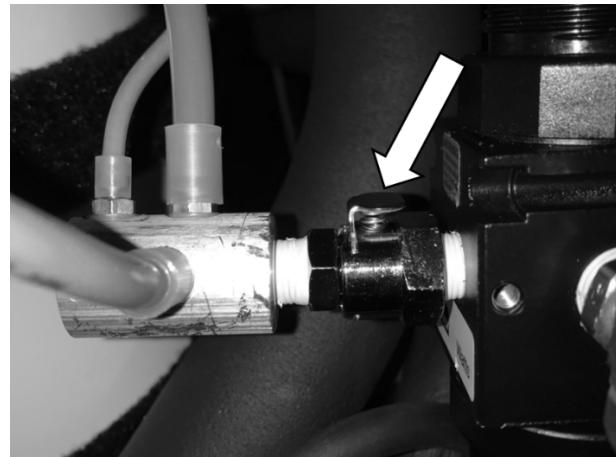
Fig. 3 – Manifold Quick-Disconnect

- 6 On the new regulator reconnect the manifold by inserting the manifold plug into the quick-disconnect socket. Then reconnect the tubing back into the push-to-connect fittings.
- 7 Slide the top of the regulator back through the mounting bracket and secure it using the plastic nut.

WARNINGS/CAUTIONS

In addition to observing the normal precautions associated with standard dental practices and procedures, the following additional precautions should be strictly noted and observed during the set-up, operation, and maintenance of this system.

⚠ WARNING

QUALIFIED PERSONNEL ONLY

The product should only be operated by qualified personnel only. The operator bears responsibility for the correct settings and proper use of the system. ASI Dental (ASI) cannot be held liable for any malfunction of this product, or performance failure and/or its designed or desired utility, nor can ASI be held liable for injuries to persons or animals, in any case when the device is misused or not operated, applied or maintained in strict accordance with user/owner instructions set out in the operation manual. In the event of any doubt or question, the user is to contact ASI for clarification or assistance.

Improperly maintained or operated systems or instruments may void the associated warranties.

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MASTER AIR REGULATOR WITH PURGE & MANIFOLD

ASI Technical Guideline for 95-0178C

65-0189 | Rev A1 | 05/19/23 | Page 4 of 4

⚠️ WARNING

COMPRESSED AIR

The compressed air system that operates this unit is under pressure. Compressed air can propel dust or loose particles and can cause bodily injury or damage. Always turn the system off and bleed off air pressure before attaching or removing air lines or accessories or servicing this unit. All air lines should be periodically inspected and replaced if worn or damaged.

If an outside compressed air supply is used to power this unit, the air supply must be regulated to 80 psi or below. Excessive air pressure could cause certain components to rupture.

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⚠️ WARNING

ELECTRICAL VOLTAGE

This system is powered by high voltage electricity. Like any other electrically powered device, if it is not used properly, it can cause electrical shock. Always plug the power cord into an electrical outlet with adequate fuse protection and proper grounding. In the event of a short circuit, grounding reduces the risk of shock by providing an escape wire for the electric current. Improper grounding of the unit can result in a risk of electric shock. Always unplug the unit before doing any service or repair to the unit.

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⚠️ WARNING

PRESENCE OF HEAVY METALS/AMALGAM

This system may be equipped with optional suction instruments. As part of dental procedures, particles of amalgam may be suctioned into the dental suction handpieces and collected within the system and trap filter.

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⚠️ WARNING

INFECTIOUS MATERIALS

Infectious disease workplace safety protocols to safeguard against cross contamination of infectious disease should always be observed. When maintaining the suction system or emptying the contents of the suction waste container, safe precautions and practices including the wearing of face mask, eye protection and gloves are to be followed.

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⚠️ CAUTION

CAUSTIC OR CORROSIVE CHEMICALS

The irrigation system may contain chemicals that are caustic or corrosive. Always verify with the operator of the system to determine which chemicals they are using within the system. Always follow workplace safety protocols when doing maintenance or service of the system including the wearing of face mask, eye protection and gloves for proper protection of exposed skin, breathing and eyes.

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