# RESET INDENTED INSTRUMENT PANEL BUTTON

## ASI TECHNICAL GUIDELINE TO RESET INDENTED INSTRUMENT PANEL BUTTON

65-0231 | Rev A1 | 07/17/23 | Page 1 of 3

### PURPOSE

This technical guideline shows how to reset an indented (stuck) instrument panel button.

### TOOLS REQUIRED

- 1/4" nut driver

#### IMPORTANT!

**Before commencing on the following instructions, please read and follow all applicable warnings/cautions listed at the end of this technical guideline.**

### INSTRUCTIONS

- 1 Turn off main power to the unit.
- 2 Open the cover to the delivery unit per the operating instructions for the model of the unit. See ASI Technical Guideline, "Delivery System Internal Access" (65-0229) to aid in opening the delivery system.
- 3 Locate four nuts on the control/button board. Untighten using a 1/4" nut driver. (Fig. 1)

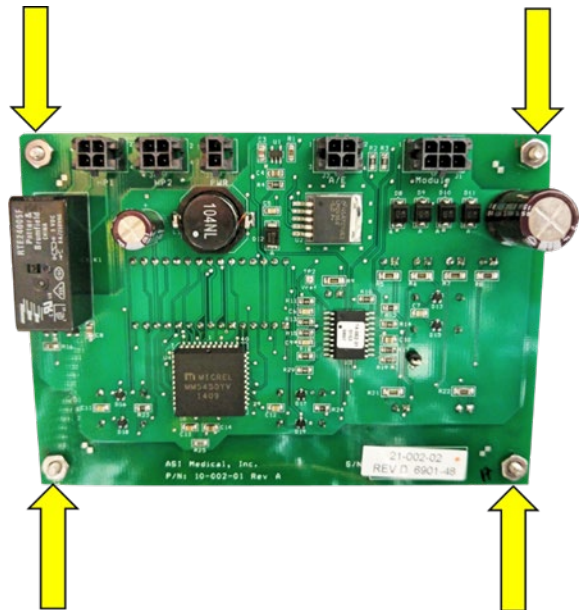
Fig. 1

- 4 Pull control board away from the plastic overlay on the front of the adapter panel. This should free up the button(s) if sticking.

**i** *The nuts that hold the control board do not need to be removed completely.*

# RESET INDENTED INSTRUMENT PANEL BUTTON

## ASI Technical Guideline to Reset Indented Instrument Panel Button

65-0231 | Rev A1 | 07/17/23 | Page 2 of 3

- 5 With the board pulled away, press the button(s) to test for function. If no function, proceed to troubleshooting the instrument using applicable ASI Technical Guideline or instrument manual.
- 6 With proper function re-established, reattach panel using Steps 2 through 4 in reverse order.
- 7 Restore power to unit and test for proper function.

### WARNINGS/CAUTIONS

In addition to observing the normal precautions associated with standard dental practices and procedures, the following additional precautions should be strictly noted and observed during the set-up, operation, and maintenance of this system.

#### WARNING

##### QUALIFIED PERSONNEL ONLY

The product should only be operated by qualified personnel only. The operator bears responsibility for the correct settings and proper use of the system. ASI Dental (ASI) cannot be held liable for any malfunction of this product, or performance failure and/or its designed or desired utility, nor can ASI be held liable for injuries to persons or animals, in any case when the device is misused or not operated, applied or maintained in strict accordance with user/owner instructions set out in the operation manual. In the event of any doubt or question, the user is to contact ASI for clarification or assistance.

Improperly maintained or operated systems or instruments may void the associated warranties.

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#### WARNING

##### COMPRESSED AIR

The compressed air system that operates this unit is under pressure. Compressed air can propel dust or loose particles and can cause bodily injury or damage. Always turn the system off and bleed off air pressure before attaching or removing air lines or accessories or servicing this unit. All air lines should be periodically inspected and replaced if worn or damaged.

If an outside compressed air supply is used to power this unit, the air supply must be regulated to 80 psi or below. Excessive air pressure could cause certain components to rupture.

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# RESET INDENTED INSTRUMENT PANEL BUTTON

ASI Technical Guideline to Reset Indented Instrument Panel Button

65-0231 | Rev A1 | 07/17/23 | Page 3 of 3

## WARNING

### **ELECTRICAL VOLTAGE**

This system is powered by high voltage electricity. Like any other electrically powered device, if it is not used properly, it can cause electrical shock. Always plug the power cord into an electrical outlet with adequate fuse protection and proper grounding. In the event of a short circuit, grounding reduces the risk of shock by providing an escape wire for the electric current. Improper grounding of the unit can result in a risk of electric shock. Always unplug the unit before doing any service or repair to the unit.

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