PURPOSE

This technical guideline provides information on how to upgrade the master air regulator, by uninstalling it from between the two air tanks and moving it closer to the back panel for easier access and serviceability.

PARTS INCLUDED

- Master air regulator (95-0178C), (1)
- Air tank drain valve (1)
- Regulator mounting bracket (1)
- Brass tee assembly (1)
- 12mm Nickel hex nut (1)
- 10-32 plugs with nylon washers (2)
- Fitting, 10-32 x 1/4" push-to connect (2)
- Nylon tubing:
 - 1/4" x 22" (2)
 - 1/4" x 8" (1)
- 1/4" x 20" air tubing with coupler (1)
- Tubing sleeves:
 - Small (1)
 - Large (3)

TOOLS REQUIRED

- Screwdrivers:
 - Phillips #2 Stubby
 - Flat head, small and medium
 - Wrenches:
 - 1/4"
 - 7/16"
 - 3/4"
 - 1/4" nut driver (deep)
 - Drill with 1/2" drill bit*
- *if applicable, refer to step 7

IMPORTANT!

Before commencing on the following instructions, please read and follow all applicable warnings/cautions listed at the end of this technical guideline.

INSTRUCTIONS

- 1 Unplug the power cord to the unit and depressurize the system.
- 2 Follow removal instructions described in technical guideline, "*Suction Waste Canister*" (65-0109), to remove the suction canister.
- 3 To gain access to the screws holding the air tank, remove the bowl and filter from the master regulator.

- 4 Using a short, stubby Phillips screwdriver, remove the three (3) screws from underneath the left air tank.

Fig. 1

- 5 Once screws have been removed, firmly grasp the regulator and pull to the left until it slides out of the air tanks.

Fig. 2

Grasp regulator and pull out of air tanks.

- 6 Install new brass tee assembly
- Apply provided silicon grease to each side of brass tee assembly.
 - Slide the brass fitting on the tee assembly into the left air tank until it completely seats.
 - Repeat on the right side until it completely seats.
- Hand start all three (3) screws that go into the bottom of the left air tank, then firmly tighten with Phillips screwdriver.

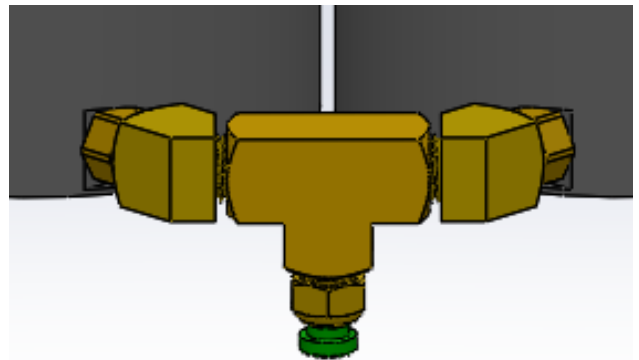
Fig. 3
Inserting new brass tee assembly

7 Installing an air tank drain valve

- Locate the rear door of the cart and check to see if there is a plug installed, then refer to the following instructions accordingly
- On the rear door of the cart, locate a plug $\sim 1/2"$ in diameter (refer to **A** in Fig. 6) and remove it. If the cart does not have a plug/hole, perform the following:
 - Make note of the new hole's location, which should be $\sim 4 1/2"$ from the left of the right corner and $\sim 10 1/2"$ from the bottom of the plate (shown as **A** in Fig. 6).
 - Using a power drill and a $1/2"$ drill bit, create a new hole that is $\sim 1/2"$ in diameter
- Install the air tank drain valve at the hole where the previous plug was located
 - Ensure the valve is on the outside of the panel pointing down (refer to Fig. 4)
- Secure the valve with the 12mm Nickel hex nut
- Install one (1) 10-32 $1/4"$ push-to-connect fitting into the back side of the valve
- Apply one (1) Purge Valve Label (61-0247) to the outside of the door directly above the purge valve

Fig. 5

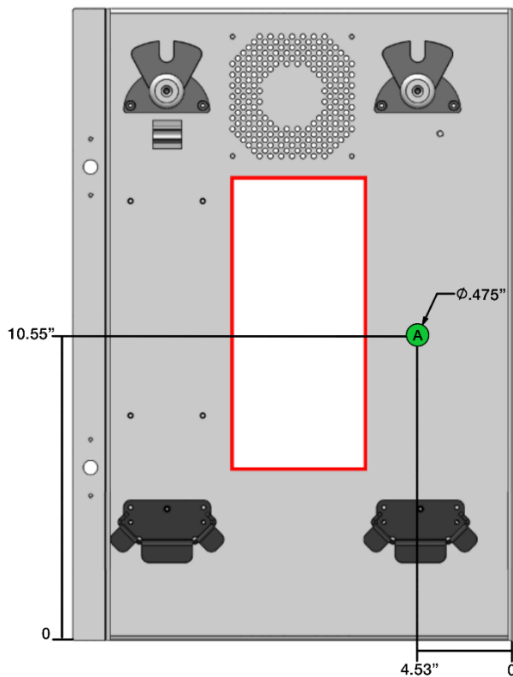
Fig. 6

Locate and remove the plug from the back panel

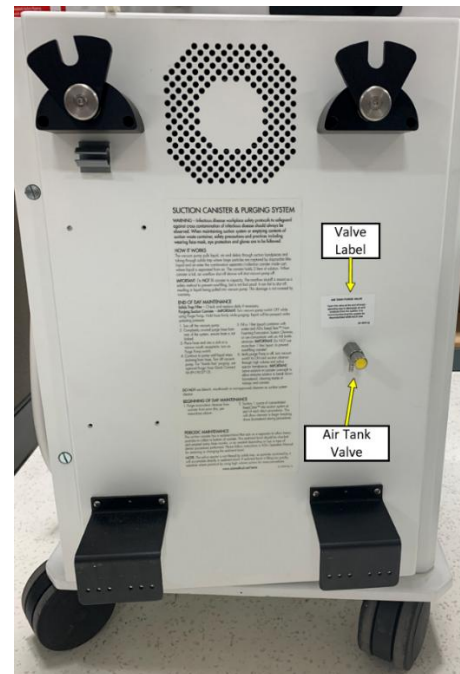
Fig. 7

Install the air tank drain valve from the outside

- 8 Remove the tubing and sleeves on the right air tank connected to the three (3) barbs.
Then remove the three (3) barbs on the right air tank using $\frac{1}{4}$ " nut driver.

Fig. 8
Remove barbs on air tank

- 9 Install the two (2) 10-32 plugs into the top two holes with the nylon washers, tighten with a $\frac{1}{4}$ " nut driver.
- 10 Install one (1) 10-32 x $\frac{1}{4}$ " push-to-connect fitting in the bottom hole, tightening with a $\frac{7}{16}$ " wrench.

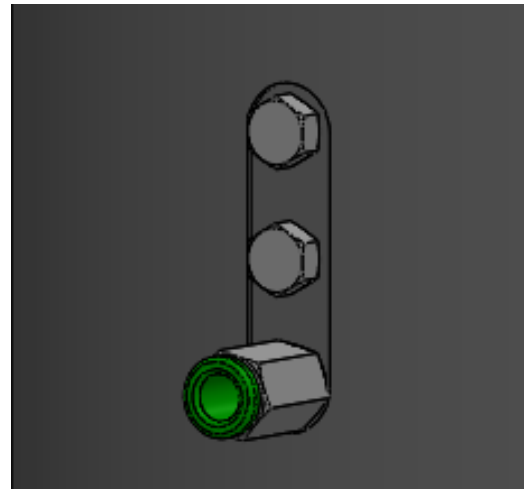
Fig. 9
Right air tank with plugs and $\frac{1}{4}$ " PTC fitting

- 11 Install the regulator mounting bracket by adhering it to the back of the water bottle housing near the ground stud; see Fig. 10.

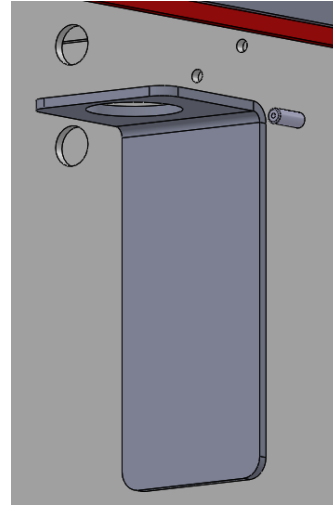
Fig. 10

Regulator mounting bracket

- 12 Mount the regulator onto the bracket, securing it with one (1) mounting nut provided.

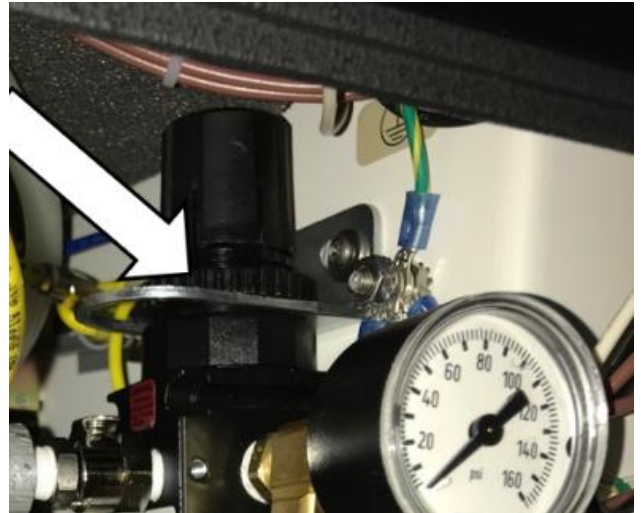
Fig. 11

Mounting regulator with manifold

- 13 Connect the air line from the closed water system (previously connected to air tank small barb) to the small barb on the regulator manifold and secure with a small sleeve.
- 14 Take the ribbed foot control tube (previously connected to the large barb on air tanks) and attach tubing extension, securing it with a large sleeve.

- 15 Connect the other end to a large barb on the manifold using one (1) large sleeve.
- 16 Attach the air supply tube (previously connected to large barb on air tanks) and connect to the top large barb on the manifold using one (1) large sleeve.

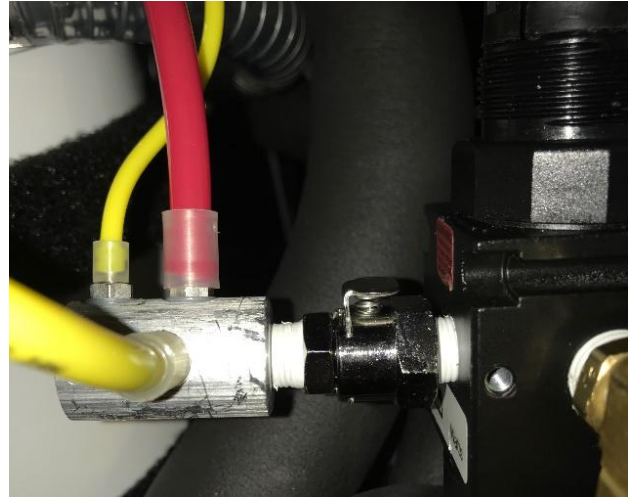
Fig. 12

Connecting air tank tubing to new regulator manifold

- 17 Take the 8" piece of nylon tubing.
Connect to the right air tank output push-to-connect fitting and the regulator input push-to-connect fitting (facing the air tanks).

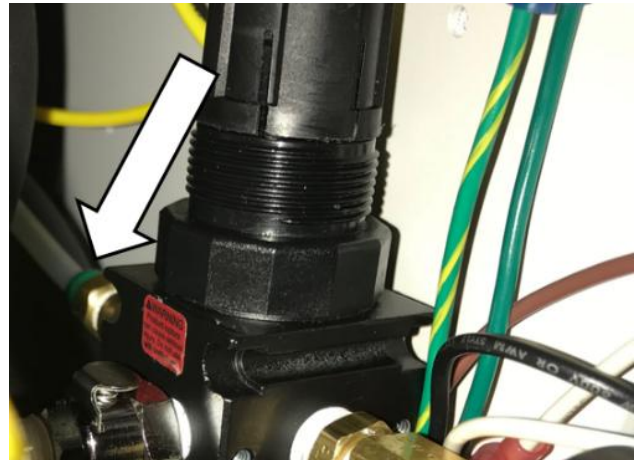
Fig. 13

Connecting tubing to the regulator input

- 18 Take the two (2) 22" pieces of nylon tubing and connect to the push connect fittings on the regulator bowl.
 - Connect one to the brass tee push-to-connect fitting on the air tanks (installed in step 6).
 - Connect the other to the push-to-connect fitting on the drain valve on the back panel (installed in step 7).

Fig. 14

Connecting tubing to regulator drain

Fig. 15

Connecting drain tube to drain valve on back panel

- 19 Replace the suction canister following replacement instructions described in technical guideline, 'Suction Waste Canister' (65-0109).
- 20 Ensure the absence of all air leaks, turn on system and power up the air compressor.
- 21 If necessary, adjust to 75-80 psi by lifting the cap on the regulator and turning clockwise to increase or turning counterclockwise to decrease

WARNINGS/CAUTIONS

In addition to observing the normal precautions associated with standard dental practices and procedures, the following additional precautions should be strictly noted and observed during the set-up, operation, and maintenance of this system.

WARNING

QUALIFIED PERSONNEL ONLY

The product should only be operated by qualified personnel. The operator bears responsibility for the correct settings and proper use of the system. ASI Dental (ASI) cannot be held liable for any malfunction of this product, or performance failure and/or its designed or desired utility, nor can ASI be held liable for injuries to persons or animals, in any case when the device is misused or not operated, applied or maintained in strict accordance with user/owner instructions set out in the operation manual. In the event of any doubt or question, the user is to contact ASI for clarification or assistance.

Improperly maintained or operated systems or instruments may void the associated warranties.

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WARNING

COMPRESSED AIR

The compressed air system that operates this unit is under pressure. Compressed air can propel dust or loose particles and can cause bodily injury or damage. Always turn the system off and bleed off air pressure before attaching or removing air lines or accessories or servicing this unit. All air lines should be periodically inspected and replaced if worn or damaged.

If an outside compressed air supply is used to power this unit, the air supply must be regulated to 80 psi or below. Excessive air pressure could cause certain components to rupture.

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WARNING

ELECTRICAL VOLTAGE

This system is powered by high voltage electricity. Like any other electrically powered device, if it is not used properly, it can cause electrical shock. Always plug the power cord into an electrical outlet with adequate fuse protection and proper grounding. In the event of a short circuit, grounding reduces the risk of shock by providing an escape wire for the electric current. Improper grounding of the unit can result in a risk of electric shock. Always unplug the unit before doing any service or repair to the unit.

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⚠ WARNING

PRESENCE OF HEAVY METALS/AMALGAM

This system may be equipped with optional suction instruments. As part of dental procedures, particles of amalgam may be suctioned into the dental suction handpieces and collected within the system and trap filter.

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⚠ WARNING

INFECTIOUS MATERIALS

Infectious disease workplace safety protocols to safeguard against cross contamination of infectious disease should always be observed. When maintaining the suction system or emptying the contents of the suction waste container, safe precautions and practices including the wearing of face mask, eye protection and gloves are to be followed.

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⚠ CAUTION

CAUSTIC OR CORROSIVE CHEMICALS

The irrigation system may contain chemicals that are caustic or corrosive. Always verify with the operator of the system to determine which chemicals they are using within the system. Always follow workplace safety protocols when doing maintenance or service of the system including the wearing of face mask, eye protection and gloves for proper protection of exposed skin, breathing and eyes.

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⚠ CAUTION

CORRECT OPERATING VOLTAGE

Check the correct operating voltage on the rating plate and ensure that the product is correctly configured for the voltage before plugging in and energizing.

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⚠ CAUTION

DAMAGE FROM UNSUITABLE ACCESSORIES

Responsibility for the use of accessories, parts or assemblies from other manufacturers rest solely with the user.

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