Electronic IFU available at: ASIsupport.com

Search by product name or part number to access the latest version of this document.

PURPOSE / OVERVIEW

This technical guideline provides information on how to upgrade the master air regulator, by uninstalling it from between the two air tanks and moving it closer to the back panel for easier access and serviceability.

PARTS INCLUDED

Part Number	Description	Qty
95-0178C	Master air regulator	1
15-0290	Air tank drain valve	1
15-0150	Regulator mounting bracket	1
23-0003	Brass tee assembly	1
15-0080	12mm Nickel hex nut	1
20-0033 & 20-0028	10-32 plugs with nylon washers	2
23-0082	Fitting, 10-32 x 1/4" push-to connect	2
22-0117	Nylon tubing, 1/4" x 22"	2
22-0117	Nylon tubing, 1/4" x 20" air tubing with coupler	1
22-0117	Nylon tubing, 1/4" x 8"	1
20-0029	Tubing Sleeve, Small	1
20-0030	Tubing Sleeve, Large	3

TOOLS REQUIRED

- Phillips #2 Stubby
- Flat head, small and medium
- Wrenches: 1/4", 7/16", 3/4"
- 1/4" nut driver (deep)
- Drill with 1/2" drill bit*

*if applicable, refer to step 7

INSTRUCTIONS

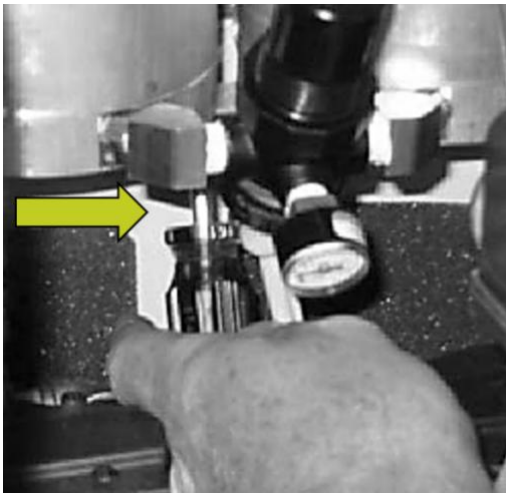
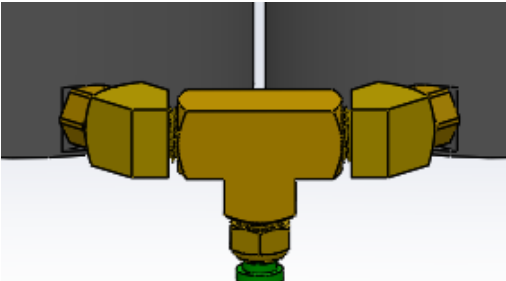
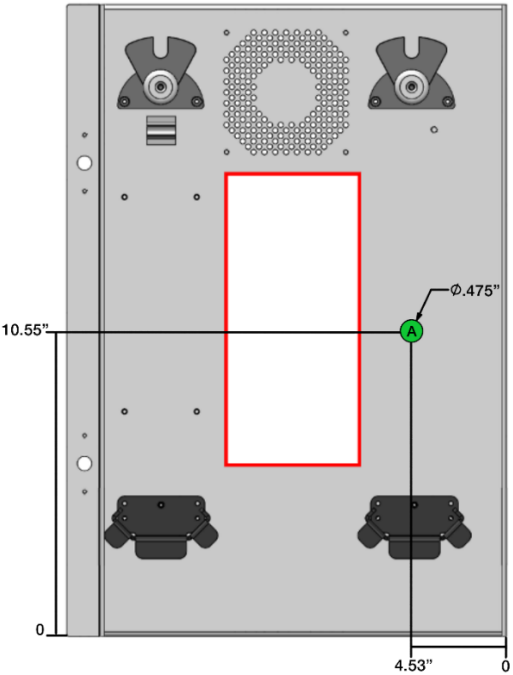
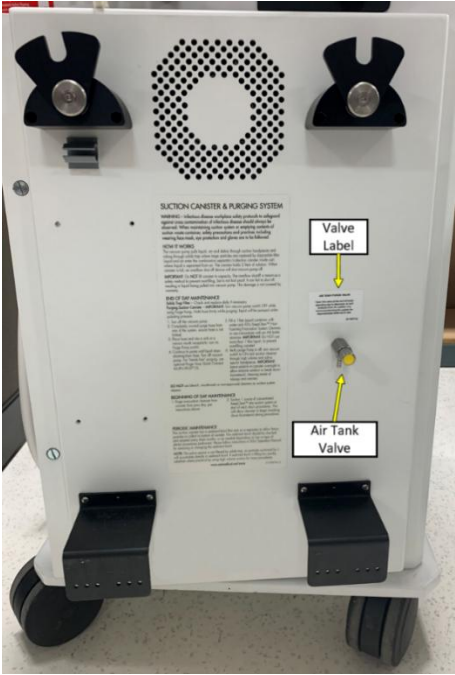
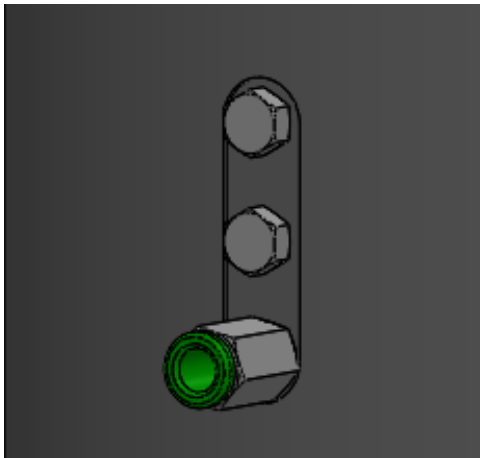
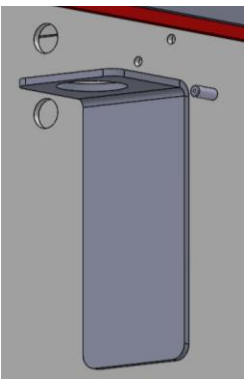
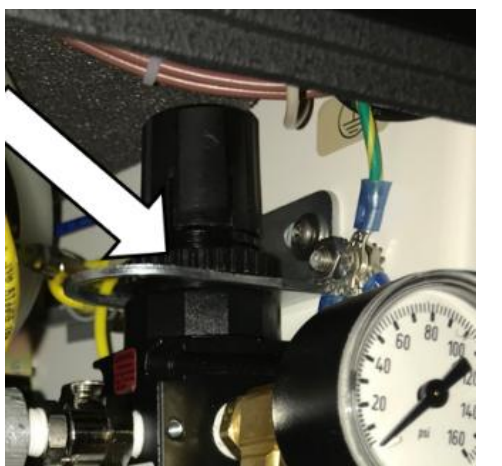
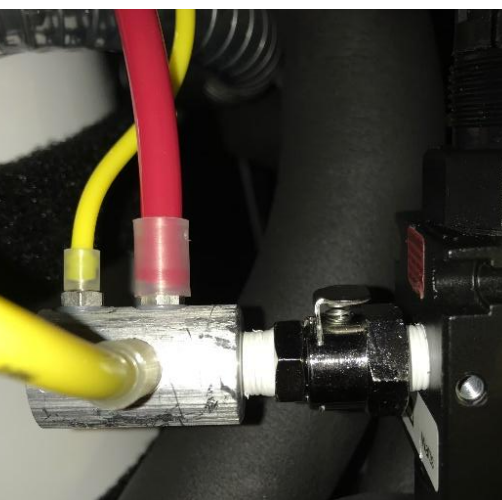
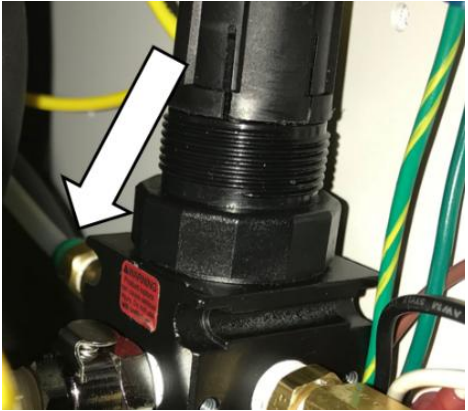
1	Unplug the power cord to the unit and depressurize the system.	
2	Follow removal instructions described in technical guideline, <i>"Suction Waste Canister"</i> (65-0109), to remove the suction canister.	
3	To gain access to the screws holding the air tank, remove the bowl and filter from the master regulator.	
4	Using a short, stubby Phillips screwdriver, remove the three (3) screws from underneath the left air tank. (Fig. 1)	
5	Once screws have been removed, firmly grasp the regulator and pull to the left until it slides out of the air tanks.	
6	Install new brass tee assembly • Apply provided silicon grease to each side of brass tee assembly. • Slide the brass fitting on the tee assembly into the left air tank until it completely seats. • Repeat on the right side until it completely seats.	

Fig. 3 – Inserting new brass tee assy.

	Hand start all three (3) screws that go into the bottom of the left air tank, then firmly tighten with Phillips screwdriver. (fig. 3)
7	Installing an air tank drain valve • Locate the rear door of the cart and check to see if there is a plug installed, then refer to the following instructions accordingly • On the rear door of the cart, locate a plug $\sim 1/2$" in diameter (refer to A in Fig. 2) and remove it. If the cart does not have a plug/hole, perform the following: ○ Make note of the new hole's location, which should be $\sim 4 1/2$" from the left of the right corner and $\sim 10 1/2$" from the bottom of the plate (shown as A in Fig. 2). ○ Using a power drill and a $1/2$" drill bit, create a new hole that is $\sim 1/2$" in diameter • Install the air tank drain valve at the hole where the previous plug was located ○ Ensure the valve is on the outside of the panel pointing down (Fig. 1) • Secure the valve with the 12mm Nickel hex nut • Install one (1) 10-32 $1/4$" push-to-connect fitting into the back side of the valve • Apply one (1) Purge Valve Label (61-0247) to the outside of the door directly above the purge valve (fig. 7)  Fig. 4
	 Fig. 2 Locate and remove the plug from the back panel
	 Fig. 3 Install the air tank drain valve from the outside

8	Remove the tubing and sleeves on the right air tank connected to the three (3) barbs. Then remove the three (3) barbs on the right air tank using $\frac{1}{4}$" nut driver.	 Fig. 4 Remove barbs on air tank
9	Install the two (2) 10-32 plugs into the top two holes with the nylon washers, tighten with a $\frac{1}{4}$" nut driver.	
10	Install one (1) 10-32 x $\frac{1}{4}$" push-to-connect fitting in the bottom hole, tightening with a $\frac{7}{16}$" wrench.	 Fig. 5 Right air tank with plugs and $\frac{1}{4}$" PTC fitting
11	Install the regulator mounting bracket by adhering it to the back of the water bottle housing near the ground stud; see Fig. 6.	 Fig. 6 Regulator mounting bracket

12	Mount the regulator onto the bracket, securing it with one (1) mounting nut provided. (fig. 11)	 Fig. 7 Mounting regulator with manifold
13	Connect the air line from the closed water system (previously connected to air tank small barb) to the small barb on the regulator manifold and secure with a small sleeve.	
14	Take the ribbed foot control tube (previously connected to the large barb on air tanks) and attach tubing extension, securing it with a large sleeve.	
15	Connect the other end to a large barb on the manifold using one (1) large sleeve.	
16	Attach the air supply tube (previously connected to large barb on air tanks) and connect to the top large barb on the manifold using one (1) large sleeve. (fig. 12)	 Fig. 8 Connecting air tank tubing to new regulator manifold

17	Take the 8" piece of nylon tubing. Connect to the right air tank output push-to-connect fitting and the regulator input push-to-connect fitting (facing the air tanks). (Fig. 13)	 Fig. 9 Connecting tubing to the regulator input
18	Take the two (2) 22" pieces of nylon tubing and connect to the push connect fittings on the regulator bowl. (Fig. 14) Connect one to the brass tee push-to-connect fitting on the air tanks (installed in step 6). Connect the other to the push-to-connect fitting on the drain valve on the back panel (installed in step 7). (Fig. 15)	 Fig. 10 Connecting tubing to regulator bowl
19	Connect the other to the push-to-connect fitting on the drain valve on the back panel (installed in step 7). (Fig. 15)	 Fig. 11 Connecting drain tube to drain valve on back panel

20	Replace the suction canister following replacement instructions described in technical guideline, 'Suction Waste Canister' (65-0109).
21	Ensure the absence of all air leaks, turn on system and power up the air compressor.
22	If necessary, adjust to 75-80 psi by lifting the cap on the regulator and turning clockwise to increase or turning counterclockwise to decrease

CAUTIONS / WARNINGS

In addition to observing the normal precautions associated with standard dental practices and procedures, the following additional precautions should be strictly noted and observed during the set-up, operation, and maintenance of this system.

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QUESTIONS OR ASSISTANCE

For technical support, contact ASI Dental Specialties:

Web: [ASIsupport.com](https://www.asisupport.com) | Email: ASIservice@ASIsupport.com | Phone: **303.407.6072** | Parts: [ASIparts](https://www.asiparts.com)

REVISION HISTORY (For Internal Master Part File Only)

CO #	Rev	Description of Change	Author	Approval Date
TG-950397				
CO-19-106	A	Initial Release	K. Vogel	07/15/19
CO-19-171	B	Revised regulator mounting bracket to new part that is installed with double sided adhesive tape, for easier installation in the field, instead of having to drill holes.	K. Vogel	10/01/19
65-0219				
AC-23-010	B1	Renumbered and reformatted into current tech guide style.	C. Hoganson	06/30/23
CO-24-250	C	Revised to update tubing lengths cited in steps 16 and 17.	M. Idabdellah	11/01/24
CO-26-108	D	Revised "Parts Included" listing: updated tubing lengths based on the 95-0397 rev D WI, added the air tank purge valve, added the hex nut. Corrected tubing length in step 17 (formerly step 16). Added a step for the air tank valve installation (step 7).	E. Fomin	07/28/26