Electronic IFU available at: ASIsupport.com

Search by product name or part number to access the latest version of this document.

PURPOSE / OVERVIEW

This retrofit kit installs two single-pad wire rope isolators along the sides of the GSE High-Flow Vacuum unit. The isolators prevent the vacuum assembly from leaning and making contact with the compressor or side of cart during operation, reducing vibration transmission and protecting both components.

PARTS INCLUDED

Part Number	Description	Qty
81-1007	Single-pad wire rope isolators	2
15-0290	M6 × 12 mm hex head screws	2
15-0150	Flat head 1/4"-20 × 1/2" screws	2

TOOLS REQUIRED

- Flat head screwdriver
- Phillips heads screwdriver
- 1/4" hex head driver or wrench
- 5/32" Hex Key

CAUTIONS / WARNINGS

In addition to observing the normal precautions associated with standard dental practices and procedures, the following additional precautions should be strictly noted and observed during the set-up, operation, and maintenance of this system.

INSTRUCTIONS

- Purge all waste from the system.

Remove the Vacuum Pump Assembly from the Cart

Turn off power to the system and unplug the unit from the electrical outlet.

Move the suction tube and main electrical cord clear of the back door to allow full access.

Open the back panel using a flat head screwdriver. The panel is secured by two quick-turn screws located on the back of the unit. (Fig 1.)

Fig. 1

⚠ CAUTION

Ensure the suction tube and electrical cord are fully clear of the back door before proceeding. Pinching or straining these components can cause damage.

Close the ball valve on the bottom of the sediment bowl to prevent any waste from leaking out and remove the inline purge tubing from the quick-connect fitting on the elbow for easier access.

Loosen the hose clamp on the hose barb connector of the vacuum pump. Slide the suction hose off the hose barb and move aside. (Fig. 2)

Locate the terminal strip on the right side of the chassis. Following the wiring label, disconnect the white wire from terminal strip position 4.

Locate the ground stud. Remove the nut and detach all ground wires.

⚠ IMPORTANT

When reinstalling ground wires in Step 6, the ground wire from the pan (top of the cart) must be installed first. Note this before removing wires.

Locate the relay on the left side of the chassis. Remove the black wire from the top-right terminal (position 2) on the relay.

Remove the three (3) 1/4"-20 hex head screws securing the vacuum plate to the cart chassis. (Fig 3.)

There are two Phillips head screws on the back of the mounting plate that clamp the mounting plate down. These must be loosened to allow the vacuum to slide out.

The vacuum pump assembly should now be free to be pulled from the cart.

Fig. 2

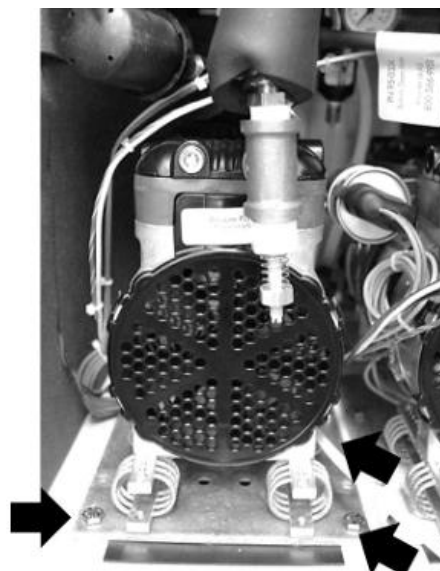
Fig. 3

2 Separate the Mounting Plate from the Wire Rope Isolators

With the vacuum pump assembly removed, loosen and remove the four (4) 1/4"-20 x 1/2" flat head screws securing the wire rope isolators to the mounting plate. Set the screws aside, as they will be reused later. Set the mounting plate aside for reinstallation in Step 5.

3	Remove the Intake-Side Wire Rope Isolator Remove the existing intake-side wire rope isolator from the assembly. This isolator will be replaced by the two single-pad isolators included in the retrofit kit.
4	Install the Two Single-Pad Isolators to the Vacuum Pump Attach both single-pad wire rope isolators to the underside of the vacuum pump using the four (4) provided M6 × 12 mm hex head screws (see Fig. 4).
5	Reinstall the Mounting Plate to the Isolators Reattach the mounting plate to the isolators using two (2) additional flat head 1/4"-20 × 1/2" screws included in the kit and screws set aside in step 2. Ensure the plate is seated flush.
6	Reinstall the Vacuum Pump Assembly into the Cart Reverse the removal procedure to reinstall the vacuum pump assembly, following the sequence below: Slide the mounting plate under the two Phillips head screws and washers in the back of the unit and secure the vacuum mounting plate to the base using three (3) 1/4"-20 hex head screws. Route the black wire to the relay and connect it to the top-right terminal (Position 2). Route the other black wire connected to the white wire to position 4 on the terminal strip ⚠ IMPORTANT Attach the ground wire from the pan (top of the cart) to the ground stud first. Then attach any remaining ground wires and secure with the nut. Slide the suction hose onto the hose barb connector on the vacuum pump and tighten the hose clamp securely. Close and secure the back panel.
7	Restore Power and Test

Fig. 4

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QUESTIONS OR ASSISTANCE

For technical support, contact ASI Dental Specialties:

Web: ASIsupport.com | Email: ASIservice@ASIsupport.com | Phone: **303.407.6072** | Parts: **ASI.parts**

REVISION HISTORY (For Internal Master Part File Only)

CO #	Rev	Description of Change	Author	Approval Date
CO-26-100	A	Release of Tech Guide	W. McPeck	08/17/26