Electronic IFU available at: ASIsupport.com

Search by product name or part number to access the latest version of this document.

PURPOSE / OVERVIEW

This technical guideline provides instructions mounting the Dentsply X-Smart Pro+ to the mounting bracket installed on the ASI delivery system.

PARTS INCLUDED

Part Number	Description	Qty
21-0159	Replacement O-Rings	1
11-0293	Replacement Springs	2
80-0319	Button Pair	1
21-0126	O-Ring Grease, Silicone	1

TOOLS REQUIRED

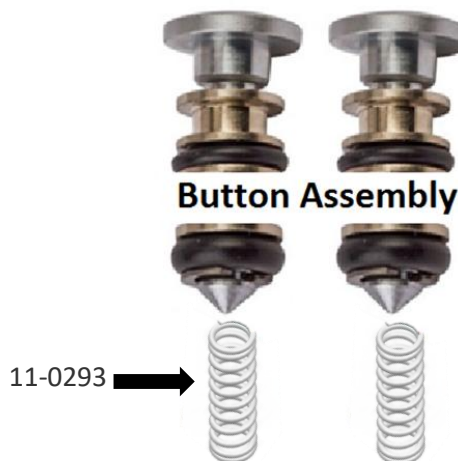
- Small Screwdriver or Punch

IMPORTANT

Before starting, please read and follow all applicable warnings/cautions listed at the end of this technical guideline.

INSTRUCTIONS

1	Air/Water Syringe Diagram 1 Syringe Adapter 2 Syringe Tip 3 Pin, Quick Clean Syringe Button 3a Air Only Syringe 3b Air/Water Syringe (11-0284) 4 Tip Adapter Assembly 5 Large O-Ring, Tip Adapter 6 Spring, Tip Adapter 7 O-Ring, Tip Adapter 8 Button Assembly 9 O-Ring, Button Assembly 10 Large O-Ring, Button Assembly 11 Retaining Ring 12 Spring, Syringe Button (11-0293) 13 8/32 Barb & Washer 14 Handle, Quick-Clean Syringe 9 and 10 come already on assembly	Fig. 1
---	---	----------------------

2	Remove the Syringe Tip (Item 2) by pressing down the retention ring located on the Tip Adapter Assembly (Item 4).
3	For Air-only Syringe service, the Pin, Quick Clean Syringe Button (Item 3a) will be located under the Syringe Adapter (Item 4) and can be easily removed by pulling it straight out. For Air/Water Syringe service, the Pin, Quick Clean Syringe Button (Item 3b) is inserted from the side and must be pushed out with a punch (11-0285).
4	Once the syringe is disassembled, start removing the O-Rings one at a time (#5 and #7 only). Carefully replace them with the ones provided, after lightly coating the new O-ring with silicone O-ring grease. Use a foam applicator for applying O-ring grease; do not use a cotton tipped applicator.
5	When reinstalling the Button Assembly, verify that the Springs are in the orientation shown in Fig. 1 so the large end is to the bottom.  Fig. 1
6	Once the Buttons are installed, press them down while reinserting the Pin, Quick Clean Syringe Button (Item 3B). Insert the Pin until it is fully seated.
7	To reinstall the Syringe Adapter (Item 1), reinsert the Spring, Tip Adapter (Item 6) and thread the Tip Adapter Assembly (Item 4) back in place. Using the Allen wrench provided, gently secure the Tip Adapter Assembly, but do not over tighten.
8	Reinsert the Syringe Tip (Item 2) and test the Syringe for proper function.

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QUESTIONS OR ASSISTANCE

For technical support, contact ASI Dental Specialties:

Web: ASIsupport.com | Email: ASIservice@ASIsupport.com | Phone: **303.407.6072** | Parts: ASI.parts

REVISION HISTORY (For Internal Master Part File Only)

CO #	Rev	Description of Change	Author	Approval Date
TG-95-0089				
A	A	Initial Release.	W. Hunsinger	11/18/16
65-0169				
AC-23-001	A1	Renumbered and reformatted to current style.	C. Hoganson	03/22/23
CO-25-011	B	Revised for clarification.	A. Lane	02/26/25
CO-25-096	C	Added 95-0556	K. Vogel	09/19/25
CO-25-175	D	Added part numbers to syringe illustration and part #'s to step 3 & 5.	S. Smith	01/26/26
CO-26-152	E	Changed the springs back to straight (11-0293)	P. Wick	09/03/26